HydroTouch - Water Soluble Lubricating Gel (STERILE)

INSTRUCTION FOR USE - Please read carefully.

DESCRIPTION

HydroTouch is a sterile, single-use syringe, pre-filled with a water-soluble gel. The lubricating gel is non-irritating, Petroleum-free Paraben-free and Chlorhexidine-free.

INGREDIENTS

100 g of **HydroTouch** contains:

Hydroxyethylcellulose, Propylene Glycol and Purified Water.

INTENDED PURPOSE

HydroTouch is a medical device that helps to prevent trauma when putting a tube or instrument into a body cavity by effective lubrication.

INDICATION FOR USE

HydroTouch is designed to lubricate the urethra prior to insertion of a urethral catheter and other urological medical devices including cystoscopes. **HydroTouch** can also be used as a lubricant gel for supra pubic catheterisation as well as for rectal and colonic applications. It is suitable for all forms of endoscopy and to prevent injuries to the colon and rectum.

HydroTouch syringes are also used in gynecological procedures for the insertion of douches, enemas, rectal thermometers and similar types of nozzles.

CONTRAINDICATIONS

HydroTouch must not be used in patients with known hypersensitivity to any of the ingredients.

BENEFITS

- Minimal risk of urethral trauma, necrosis or catheter induced urethritis.
- The risk of urinary tract infection is reduced because urethral defence mechanisms are not compromised.
- Access for cleansing and catheter changes are easier, allowing greater ability for patients to self-care. Allows greater comfort, particularly for patients who are wheelchair-bound.
- Allows the facilitation of trials of voiding after major surgery.
- The transparent gel leaves the optics of the instruments visibly clear.

WARNINGS

- Do not use if you have allergic reaction any ingredients of product.
- There is no known side effect. If you notice any side effects, please inform your doctor, pharmacist or the manufacturer.
- Do not use if package is damaged.
- Do not use after expiry date.
- For single use only. Not to be used for i/v and i/m injections.
- Keep away from children.

HOW TO USE

HydroTouch is available in two sizes. The prefilled syringes contain 6ml and 11ml gel. Usually the complete contents of the size suitable for the procedure will be used. The syringe is removed from its sterile package and a desired amount is added to the patient body or medical equipment.

STORAGE / SHELF LIFE

Store at 5-30°C until expiry date. Keep away from sunlight and keep dry. The shelf life of the product is 3 years. Do not use after expiry date. The expiration date is printed on the blister packaging and on the box. Any remaining gel in the syringe/accordion tube must be disposed according to local policies and procedures.

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4M Medical GmbH